
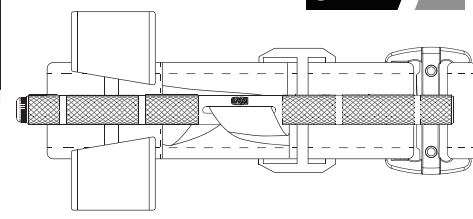
instructions
on the net

Product Code: H5000-4055G
UDI-DI: 06973105231720


CVN MEDICAL SOLUTIONS
A Part of CAREVISION Division
Hiltusentie 26 A 2, FI-90620 OULU, Finland | info@cvn.fi
CAREVISION OY NCAE A843G

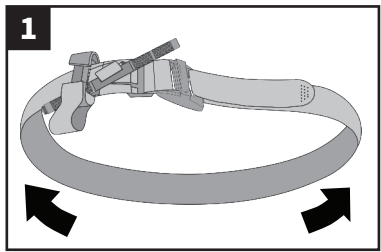
STORING THE TOURNIQUET IN THE QUICK LAUNCH CONFIGURATION

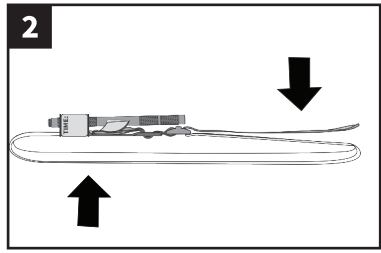Pass the **grey tip** through the slit in the buckle. Pull 8" of band through, fold it back and adhere the band to itself.

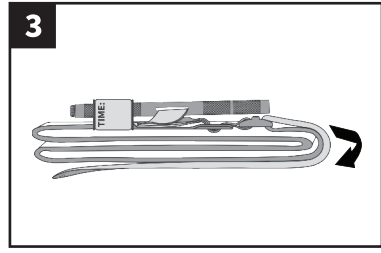
Flatten the loop formed by the band. Place the buckle in the middle of the flattened band.

Fold the tourniquet in half placing the buckle at one end. Make sure the TIME strap is open and in ready position; as shown in the illustration.

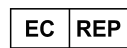ANPING GUARDIAN MEDICAL EQUIPMENT CO.,LTD
500 Meters, Nan Sucun, Anping County, Hebei, China


MedNet EC-REP GmbH
Borkstrasse 10, 48163 Münster, Germany
SRN: DE-AR-000000002
Tel: +49 251 322 66 61 Email: ecrep@medneteuropa.com


PRODUCT FEATURES:

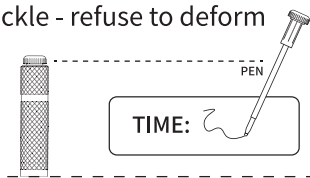1. High strength aluminum alloy rod
2. Tear resistant moving webbing
3. Aluminum parts built in the adjusting buckle - refuse to deform
4. Self-contained pen with time recording


INSTRUCTIONS FOR USE

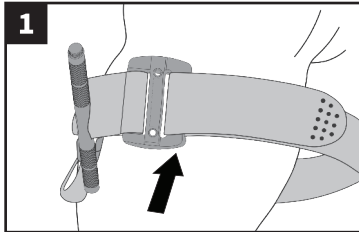ONE-HANDED APPLICATION


1 Insert the injured limb through the loop in the band and position the tourniquet 2-3" above the bleeding site directly to the skin.

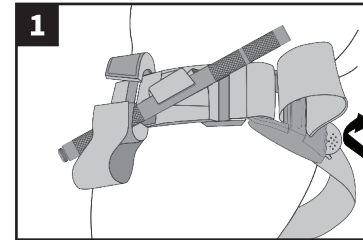OR


TWO-HANDED APPLICATION

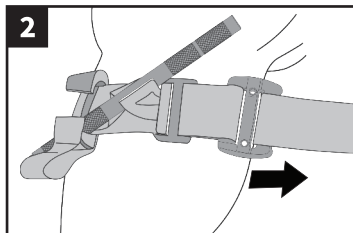
1 Route and band around the limb, and pass the tip through the gaps for the buckle. Position the tourniquet 2-3" above the wound site.

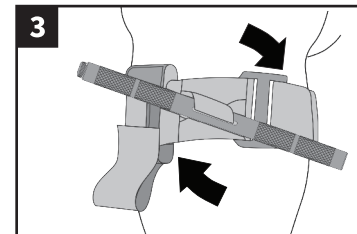
2 The band should always travel around both, the rolling pin and sliding teeth in the buckle. Pull the band tight and fasten it back on itself, all the way around the limb but not over the rod clips. The band should be tight enough so that 3 fingers cannot slide beneath it. If they can, re-tighten and secure.

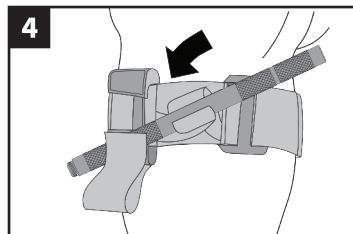3 Twist the rod until bleeding has stopped. Also check for distal pulse beneath the tourniquet near the wound, this should disappear.

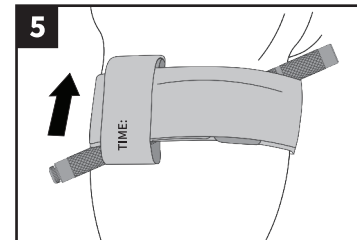
4 Secure the rod inside the clips to lock it in place. Check for bleeding and distal pulse. If bleeding continues or distal pulse is present below the tourniquet, consider additional tightening or applying a second tourniquet above the first. Reassess.


5 Route the band between the clips and over the rod. Secure the rod and band with the TIME strap. Record time of application with the pen at the top. Consider further securing with medical tape. Ensure tourniquet is visible for medical personnel.

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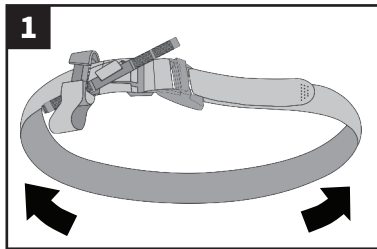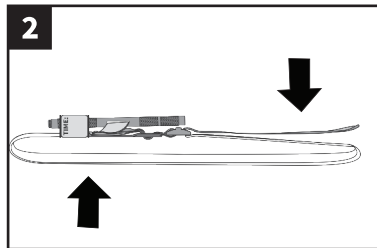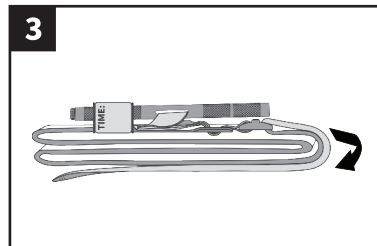
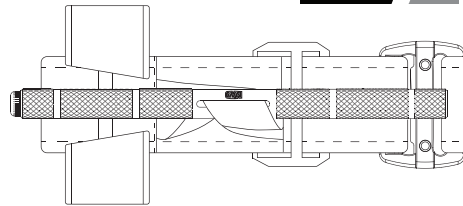
GEN 2

Product Code: H5000-4055G
UDI-DI: 06973105231720


CVN MEDICAL SOLUTIONS
A Part of CAREVISION Division
Hiltusentie 26 A 2, FI-90620 OULU, Finland | info@cvn.fi
CAREVISION OY NCAGE A843G

instructions
on the net

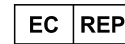ANPING GUARDIAN MEDICAL EQUIPMENT CO.,LTD
500 Meters, Nan Sucun, Anping County, Hebei, China

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
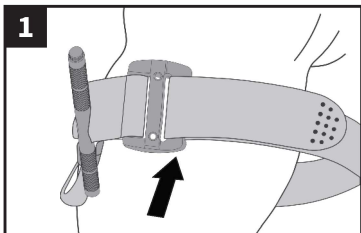
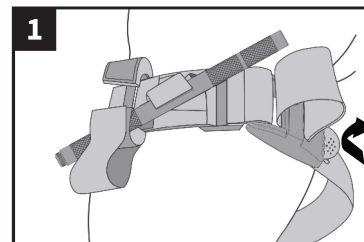
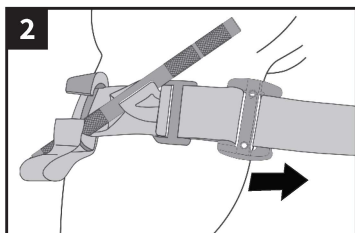
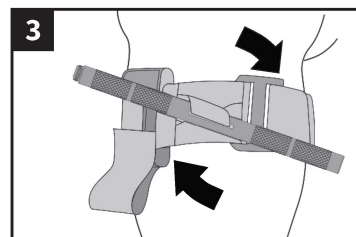
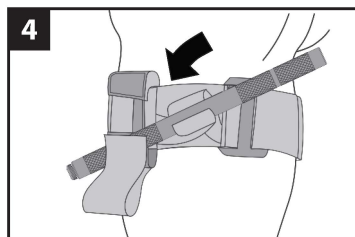
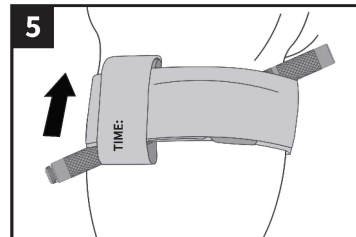
Product Code: H5000-4055G

UDI-DI: 06973105231720

CVN MEDICAL SOLUTIONS
A Part of CAREVISION Division

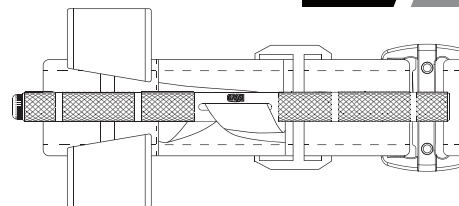Hiltusentie 26 A 2, FI-90620 OULU, Finland | info@cvn.fi
CAREVISION OY NCAE A843G

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